



DOUBLE CRANK ADJUSTABLE HEIGHT TABLE

P/N T0082-6624

USER MANUAL



Mac Medical Inc.
820 S. Mulberry St.
Millstadt, IL 62260
(618) 476-3550 phone
(618) 476-3337 fax

Adjustable Height Table

Prior to cranking the table up or down, ensure that the two caster brakes have been set.

The Adjustable height table has two (2) black crank handles, one on each end of the table.

Each crank handle can be used independently or together to crank the table up or down. Reference the directional sticker on each end of the table to determine which way to turn the crank handle for desired table height.

To use the crank handle, ensure that the handle is engaged into receiver (Reference Picture #1).

If one crank handle is to be used, ensure the handle not being used is disengaged from the receiver and folded down (Reference Picture #2).

If turning the crank does not raise or lower the table, the turning crank handle is not engaged completely. Wiggle the crank handle being used right and left slightly while also gently pushing inward toward the table. This should completely engage the crank handle.

If both crank handles are to be used, ensure that both crank handles are completely engaged into the receiver.

Each crank handle turns in the opposite direction as the other crank handle to raise and lower the table top.

WARNING – Always be careful around moving parts due to a potential pinch hazard.



Picture #1



Picture #2