

Cleaning Instructions

A: Instructions for Cleaning Standard Warming Cabinets:

Warming Cabinets must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the Warming Cabinets, make sure to use the proper cleaning agents and cleaning materials.



CAUTION: DO NOT USE the following cleaning materials on the stainless steel surfaces of the Warming Cabinets:

- Abrasive Pads
- Scrapers
- Steel Wool
- Wire Brushes
- Denatured Alcohol



CAUTION: DO NOT USE the following cleaning agents on the stainless steel surfaces of the Warming Cabinets:

- Hard Water
- Bleach or any product containing Chlorine
- Hydrochloric Acid

Use the following cleaning materials and agents:

- Soft, clean lint-free cloth
- Distilled Water
- Cleaners approved for use on stainless steel
- Distilled water with mild detergent



NOTE: Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the Warming Cabinets.

How to Clean the Warming Cabinet Interior:

1. Unplug and remove the Warming Cabinet from its power supply.



CAUTION: Turn OFF Circuit Breaker if Warming Cabinet is hard wired into the facility's electrical supply.

2. Open the door and remove all adjustable shelves and shelf clips.

3. Clean the adjustable shelves and shelf clips separately.
4. Using a damp, lint free, cloth and approved cleaner, wipe down the interior of the cabinet.
5. Use a lint free dry cloth to dry the cabinet's interior or let air dry. If air drying, ALWAYS leave the cabinet door open.

How to Clean Warming Cabinet's Exterior:

Stainless Steel:

1. Use the commercially prepared stainless steel cleaner and follow the manufacturer's directions.

Glass Door:

1. Use a commercially prepared AMMONIA free glass cleaner or use Distilled Water and a Mild Detergent applied with a lint-free cloth.

Decals or Printed Labels:

1. Use Distilled Water and a Mild Detergent applied with a lint-free cloth. DO NOT use any chemical cleaners.

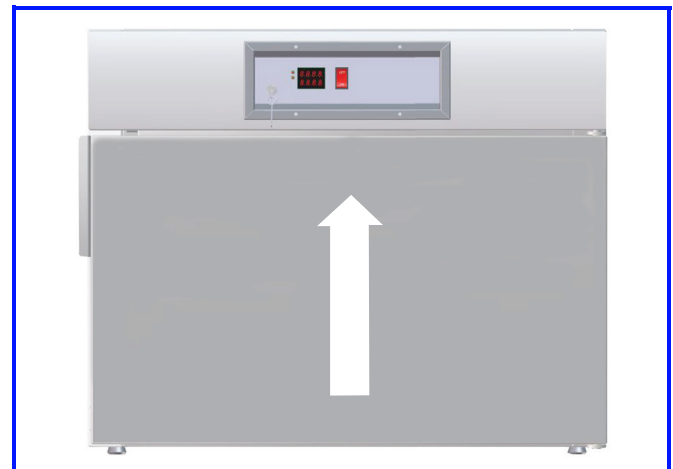


Figure 1: Cleaning Stainless Steel

2. Clean the Warming Cabinet with the grain of the Stainless Steel. See Figure 1.



CAUTION: DO NOT flood, steam clean, or hose the Warming Cabinet at any time as damage to the cabinet's surfaces can occur. These methods can also result in an Electrical Hazard.