




## USB Data Download Instructions For D-Series Warming Cabinets

1. Insert USB flash drive into USB slot under data symbol (📁) as shown below.

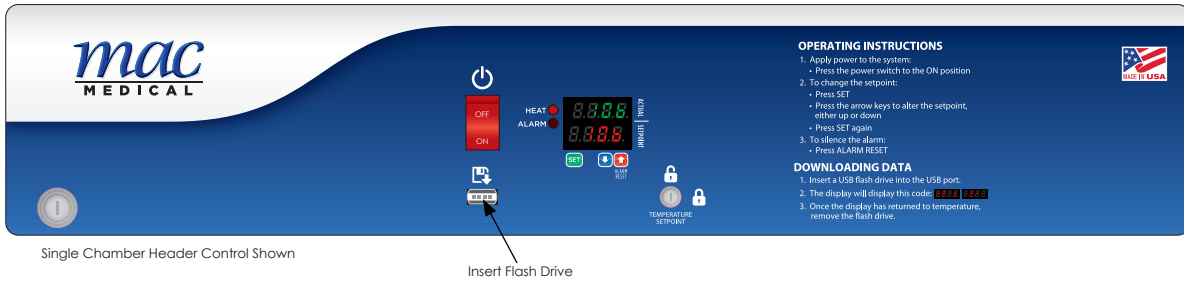

2. Data download will automatically begin.

3. During download display will read:



4. Once the display shows the temperature again, it is safe to remove the flash drive.
5. Data is saved on the flash drive as a .CSV file for easy import into most spreadsheet programs (e.g. Microsoft Excel).

### Sample Report

SWC243074 Single, S/N: 062112-D01, 031115

DATE/TIME	TEMPERATURE	°C/°F
3/10/2015 23:12	106	F
3/10/2015 23:43	106	F
3/11/2015 0:12	106	F
3/11/2015 0:43	106	F
3/11/2015 1:12	106	F

6. Data is secure (not reported or connected to any network).

MAC Medical, Inc.  
Printed in USA  
Publication No. IFU-047  
July 2015

Information regarding this product is  
subject to change without prior notice.