



Time set up for D-Series Warming Cabinets

1. Remove the top panel (4 Phillips head screws).

2. Remove the inside top panel (2 Phillips head screws).



CAUTION: Exposure to high voltage.



CAUTION: Risk of electrical shock.

3. Press and hold the round black Menu Change button (located on the back of the electrical enclosure on the right side) and the Set and Up arrow at the same time.

4. The display will then flash "AD_N". Release the Set and Up arrow but continue to hold down the Menu Change button.

5. Hit the Set button 6 times until the screen reads "Hour" on the top line.

6. Next, press the Up or Down arrow to the desired number.

7. Next, press the Set button. The display will now read:



Hit the Up or Down arrow to the desired number.

8. Release the Menu Change button and hold the Set button down until it returns to the main screen.