



INSTRUCTION FOR USE Priming Processing IR Sink Soap Pump Install Soap Pump Battery Pack

This instruction will demonstrate how to prime the soap pump, which is necessary to start the pumping of soap.

Use only non-detergent soap.

For this operation, you will need a plastic syringe provided by MAC Medical (H0905-01) and a full soap container (not supplied by MAC Medical) to place the soap pump tubing (S0095) into. This tubing is pre-installed with the sink



Place the loose end of the tubing into the soap container (not supplied by MAC Medical). Insert the syringe tip into tip of the sink soap pump head with the syringe plunger fully depressed. Withdraw the syringe plunger to suck the soap through the soap pump dispenser.



Keep priming the soap pump with the syringe until the soap begins dispensing through the soap pump head dispenser.

MAC Medical, Inc.
Printed in USA
Publication No. IFU-054 Rev A
December 2015


Information regarding this product is
subject to change without prior notice.



INSTRUCTION FOR USE Priming Processing IR Sink Soap Pump Install Soap Pump Battery Pack

An optional power source for the soap pump is the battery pack, which is to be mounted (vertically or horizontally) on the wall below the sink. Be sure to provide access to the battery pack so that batteries can be changed regularly.

The battery pack operates on four (4) D size cell batteries (not supplied by MAC Medical).

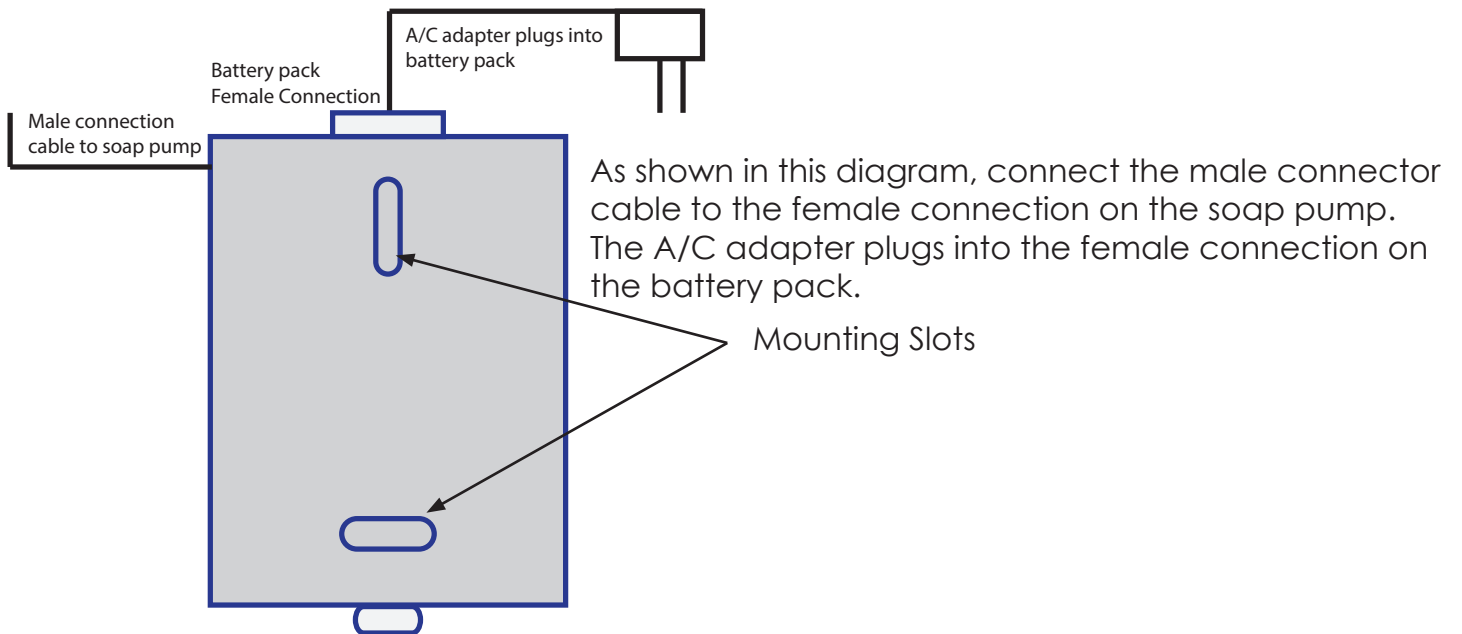

Interior of battery pack



Exterior of battery pack

MAC Medical, Inc.
Printed in USA
Publication No. IFU-054 Rev A
December 2015

Information regarding this product is subject to change without prior notice.