

Installation Instruction -

Installing the Back Bracket for Adjustable Height Prep and Pack Tables

The TWS-3060 Pack and Prep table is shipped with the back bracket in a lowered position. This is to facilitate easier shipping.

1. To raise it to its operational position, remove the eight 5/16-18 X 5/8 Hex bolts (six bolts at table level, two bolts at the bottom) that hold the back bracket in its shipping position (Fig. 1).
2. Remove the two shipping brackets. Save these in case the pack and prep table is moved to another location (Fig. 2).

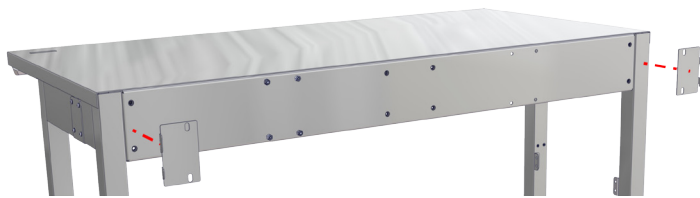


Fig. 2: Remove shipping brackets



Fig. 1: Shipping position - remove bolts

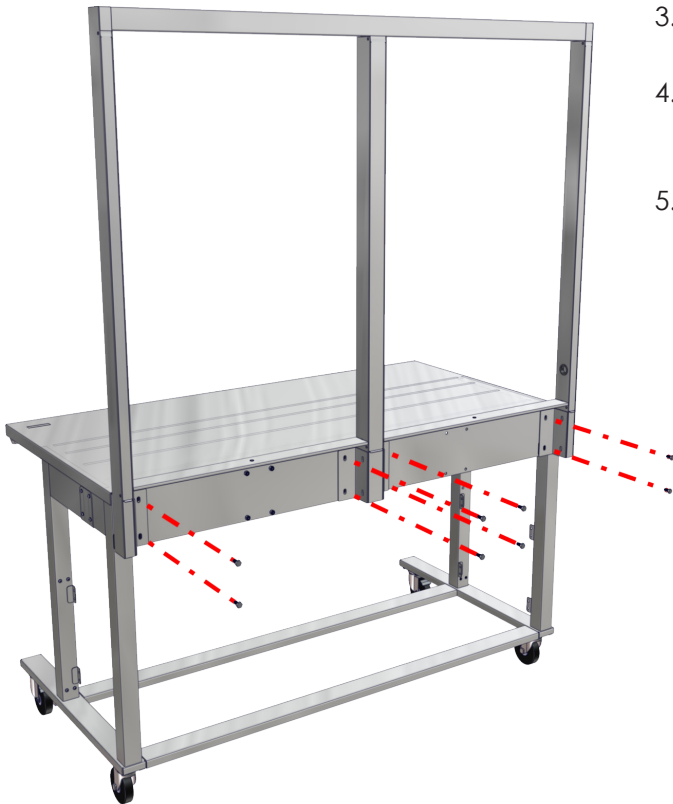


Fig. 3: Raise back bracket & insert bolts

3. Raise the back bracket and align its mounting plate holes with the table bolt holes. (Fig. 3).
4. Re-insert the bolts through the back bracket mounting plate holes (2 bolts at each end and 4 bolts in the center) and tighten (Fig. 3 - Fig. 4).
5. If the unit must be moved to a new location, remove the bolts, lower the back bracket to its shipping position, re-insert the shipping brackets and re-secure with the bolts.

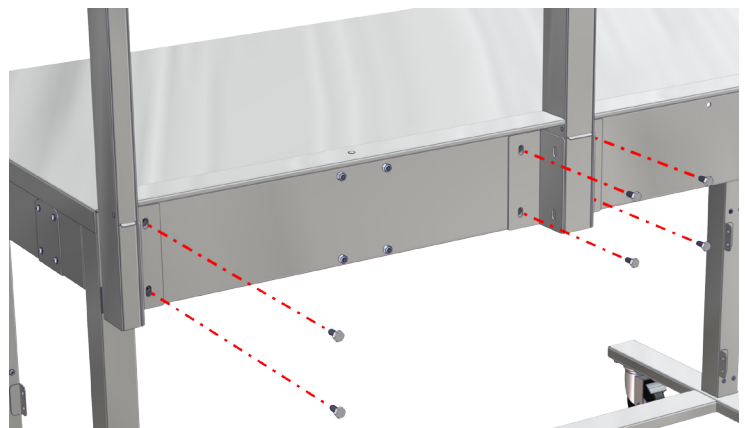


Fig. 4: Close-up of bolt re-insertion