

Instruction for Use - Installing a Paper Roller Dispenser on a Stretcher

The purpose of these instructions are to describe the installation of a Paper Roller Dispenser Kit on a MAC Medical Stretcher.

The following tools will be necessary to perform this installation:

- 1/4" Socket Wrench
- Crescent Wrench
- Flathead Screwdriver

The STR0276 Paper Roll Dispenser Kit will contain the following:

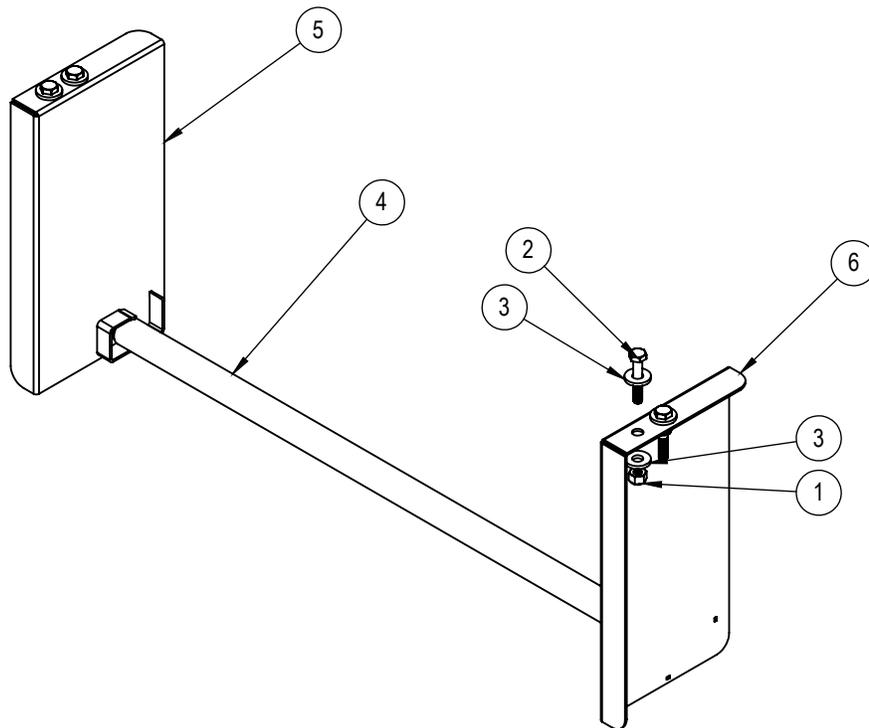


Fig. 1: Paper Roll Dispenser Assembly Kit

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	H0004-01	1/4-20 LOCK NUT	4
2	H0016-21	1/4-20 X 1 1/2 LG HEX HEAD BOLT	4
3	HDW0001-03	1/4" FLAT WASHER	8
4	STR0268-01	PAPER ROLL TUBE	1
5	STR0274L-01	PAPER ROLL BRACKET ASSEMBLY, LEFT	1
6	STR0274R-01	PAPER ROLL BRACKET ASSEMBLY, RIGHT	1

Instruction for Use - Installing a Paper Roller Dispenser on a Stretcher

1. Begin by gently removing the 4 black plastic top caps from the two Mounting brackets at the foot of the frame assembly with flathead screwdriver. (fig. 2)


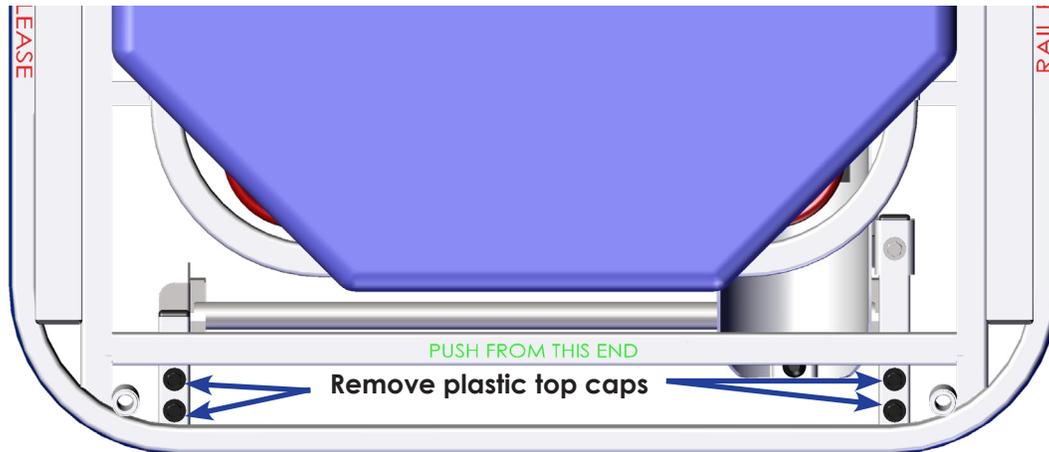


Fig. 2: Plastic Top Caps in Mounting Brackets

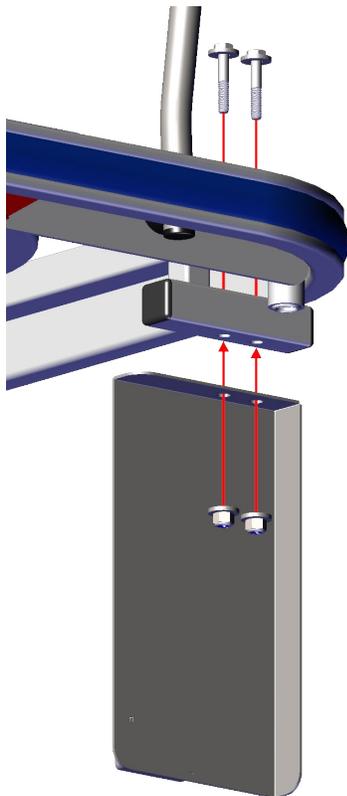

2. Insert washers through bolts. Carefully push bolts with washers through the top of the holes in the mounting bracket. (Fig. 3)
3. Slide a Paper Roller Dispenser Bracket assembly onto the bottom of the bolts (make sure the brackets on each side that hold the Paper Roll Tube are positioned facing inward). (Fig. 3)
4. From underneath, place a flat washer onto the exposed threads of the hex head bolt, then thread the 1/4" lock nut onto the bolt until the lock nut starts to lock up. Hold the lock nut in place with a crescent wrench while tightening the hex head bolt from above using a socket wrench. Ensure the bolt and nut are securely fastened. (Fig. 3)
5. Repeat steps 2-4 for the other Paper Roller Dispenser bracket.

Fig. 3: Installing the Paper Roller Dispenser bracket

Instruction for Use - Installing a Paper Roller Dispenser on a Stretcher

6. Place the roll of paper onto the assembly by inserting the roller tube ends into the inner brackets of the Paper Roller Assembly. The Paper Roll Dispenser is now ready for use (Fig. 4)

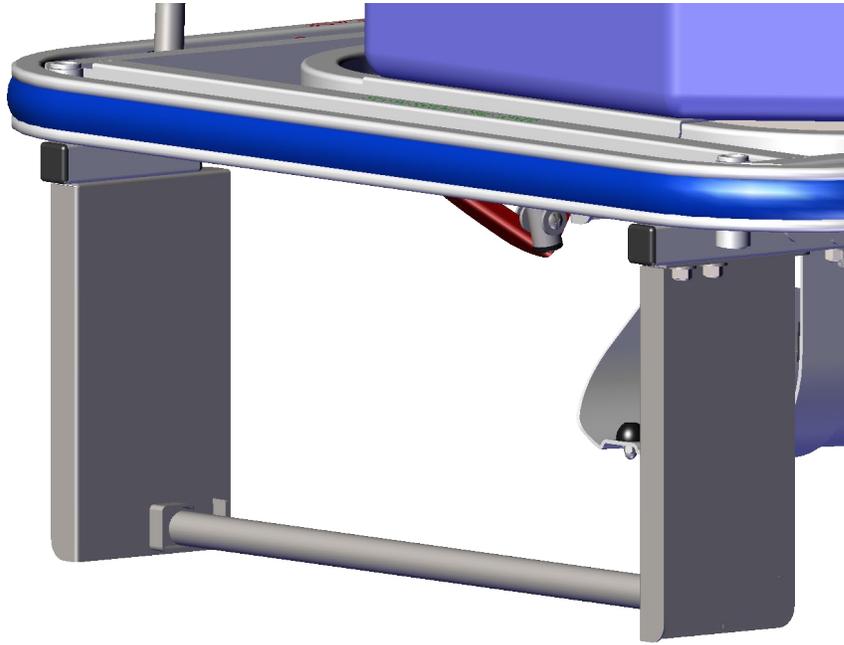


Fig. 4: Fully Installed Paper Roller Dispenser