



INSTRUCTION MANUAL



Stainless Steel Kick Bucket and Stand

Assembly, Operation and Cleaning Instructions



Read and understand all of the instructions and safety information in this manual before operating this product.

DESCRIPTION OF PRODUCT

The Products covered in this manual are the 8" and 14" Stainless Steel Kick Buckets.

PURPOSE OF THIS MANUAL

This manual is to provide the user with instructions on assembling, using and cleaning the Stainless Steel Kick Bucket.



Warning! Read and understand these instructions before assembling or using the Stainless Steel Kick Bucket and Stand Set. If the Kick Bucket and Stand Set is not properly assembled, personal injury and damage to the Kick Bucket and Stand Set could result.



Warning! If components are damaged or missing, contact your MAC Medical representative immediately. Do not use substitute parts. Use only MAC Medical replacement parts. Failure to use the correct replacement parts could lead to personal injury and damage to the Kick Bucket and Stand Set.

Standard Features

- Stainless Steel Frame
- Perimeter Bumper to Protect Walls
- Tip Resistant
- 2" Swivel Casters



KB-1000

Overall Dimensions:

14" H x 13 1/2" diameter.
Removable 13 Quart
Stainless Steel Basin.



KB-2000

Overall Dimensions:

8" H x 16 1/4" diameter.
Removable 9 1/2 Quart
Stainless Steel Basin.


KEEP THIS MANUAL

Assembly



Warning! Assembly, repair and maintenance must be performed by qualified personnel only.

1. Carefully remove the Kick Bucket and Stand from its packing material.
2. Place the Kick Bucket in the Stand as shown below and carefully press downward until the Kick Bucket is fully seated in the Stand.

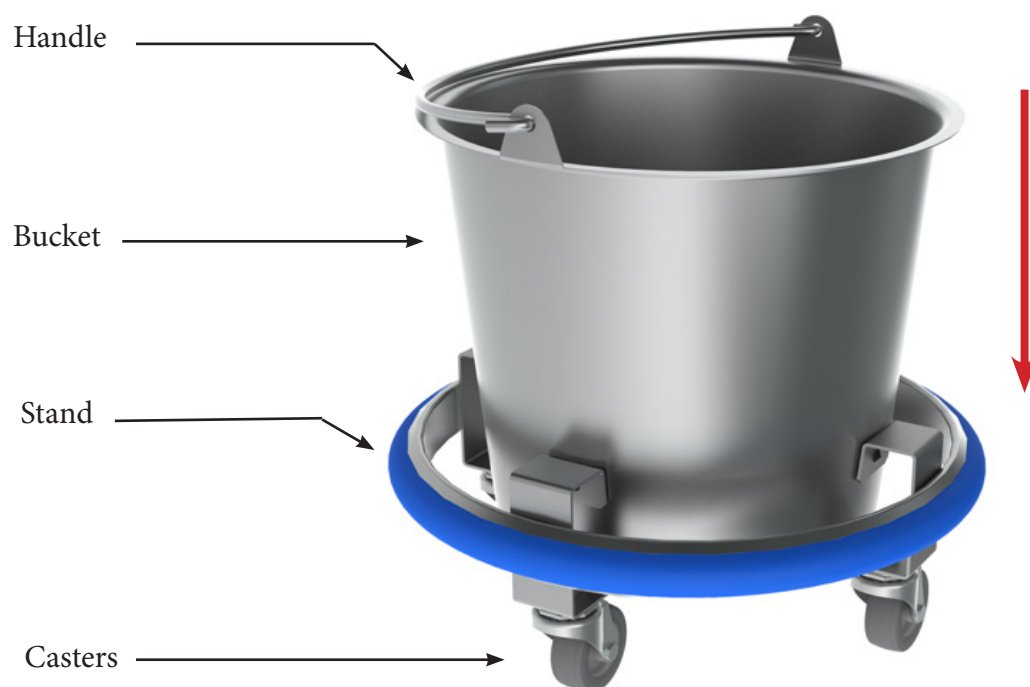

Operation

- Clean the bucket before and after each use.
- Ensure all components are secure before each use.

Casters and Wheels - Care and Maintenance

Users of the Stainless Steel Kick Bucket are responsible for the proper care, use and maintenance of the equipment. The Kick Bucket Stand can become damaged and unsafe if used improperly.

- Do not overload the Kick Bucket Stand.
- Do not drop heavy loads on the Kick Bucket.
- Do not use the Kick Bucket at high speeds.

Using the Kick Bucket in any of the above conditions can result in failure of the Casters, Wheels or other parts of the equipment.

Periodically inspect the bottom of the Kick Bucket Stand and look for any of the following issues –

- Broken welds
- Loose nuts or bolts. These should be tightened.
- Frame Distortion. This could be caused by an overloaded Bucket or by impacts with other objects while moving the Bucket.

Casters:

- Make sure the Casters are in good condition. For Swivel Casters, if the caster has excessive play due to wear, it may need to be replaced. If it does not turn freely, corrosion or dirt may also force its replacement.

Wheels:

- Wheels should be checked for visible tread wear. Flat spots due to accumulation of material, such as string or thread, can cause the wheel to bind. The axle bolt nut should be removed and the foreign material removed. If the internal components are intact, the caster can be reassembled.
- Rubber tires can be badly worn and cause erratic steering, bumping, load shifting and floor damage. Replacement wheels and bearings should be kept on hand to keep equipment in working order.

Cleaning and Maintenance

The Kick Bucket and Stand must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the Kick Bucket and Stand, make sure to use the proper cleaning agents and cleaning materials.



CAUTION: DO NOT USE the following materials on the stainless steel surfaces of the Kick Bucket:

Abrasive Pads
Scrapers
Steel Wool
Wire Brushes



CAUTION: DO NOT USE the following agents on the stainless steel surfaces of the Kick Bucket:

Hard water
Bleach or any products containing Chlorine
Hydrochloric Acid
Denatured Alcohol



NOTE: Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the Kick Bucket and Stand.

Use the following cleaning materials and agents:

Soft, clean lint-free cloth
Distilled water
Distilled water with mild detergent
Cleaners approved for use on stainless steel

How to Clean the Kick Bucket and Stand:


- Using a damp, lint-free cloth and approved cleaner, wipe all stainless steel interior and exterior surfaces of the Kick Bucket, and all surfaces of the Stand.
- Use a lint-free dry cloth to dry the Kick Bucket and Stand, or let it air dry. If air drying, make sure that the Kick Bucket and Stand are completely dry before storing, and store only in dry conditions.

Decals or Printed Labels:

- Use distilled water and a mild detergent applied with a lint-free cloth. DO NOT use any chemical cleaner.

Replacement Parts - General

The Kick Bucket replacement parts listed on this page have been identified by MAC Medical as serviceable by facility personnel and are available for purchase. To obtain MAC Medical certified parts and authorized services, contact your MAC Medical representative.

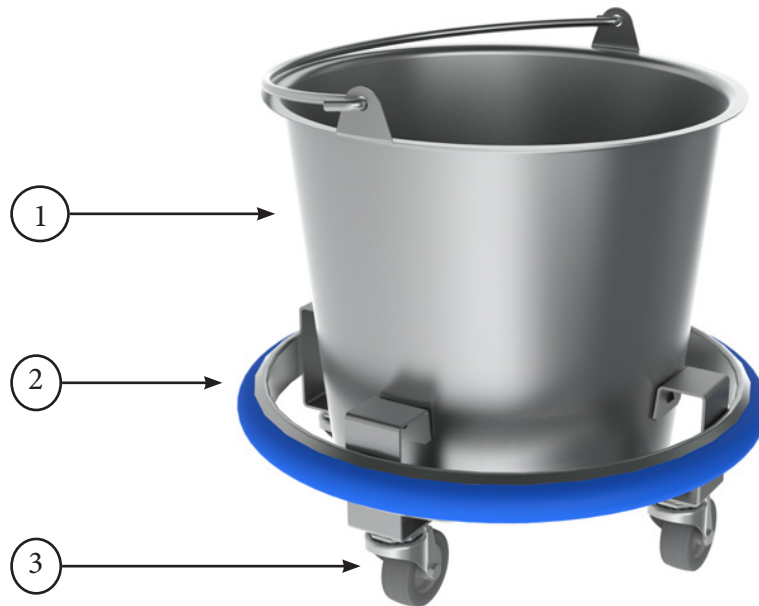

Item ID	Part Number	Description	QTY	Model
1	KB0001	13 Quart Kick Bucket	1	KB1000
1	SOL0016	9 1/2 Quart Stainless Solution Bowl	1	KB2000
2	KB0005	Kick Bucket PVC Bumper	1	KB1000
2	KB005-01	Kick Bucket PVC Bumper	1	KB2000
3	KB0006	2" Swivel Caster	4	KB1000/ KB2000

Notes

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Limited Lifetime Warranty

MAC Medical warrants to the original purchaser that its Proprietary Products will be free from defects in workmanship or materials under normal use and service for the life of the product, so long as owned by the original purchaser, according to the limitations set forth below.

Defective products shall be repaired or replaced at MAC Medical's option at no cost to the original purchaser provided:

1. The customer must obtain a written return authorization supplied by MAC Medical's customer service department. The product must be returned, properly packaged, with a copy of the original sales receipt and copy of the RMA authorization. Please call (877) 828-9975 or (618) 476-3550 to receive a return authorization.
2. The customer must pre-pay freight charges to and from MAC Medical and must ship merchandise properly packaged, in a way the product will not be damaged during transit to the factory. MAC Medical does not warranty any freight damage to or from the factory. It is the customer's responsibility to inspect the product for packaging damage before signing the BOL.

This warranty does not apply to products, which have been subject to abuse, misuse, accident, modification, alteration, tampering, negligence, lack of routine maintenance or misapplication; or products that have been repaired by other than MAC Medical or its authorized representatives.

This warranty does not apply to glass, fabrics, vinyl, seat coverings, cushions, padding or their stitching, gluing or installation. Component parts not manufactured by MAC Medical, such as casters, caster inserts, any components made from rubber or plastic, circuit boards, fan heaters, plumbing parts, electrical switches and other components are also excluded. For these component parts, MAC Medical will pass on the original manufacturer's warranty to MAC Medical equipment original purchasers. Please consult factory for questions regarding the warranties of these component parts. This warranty does not apply to custom fabricated products. Consult factory.

UNLESS EXPRESSLY SET FORTH IN THIS WARRANTY, THERE ARE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND/OR MERCHANTABILITY OR ANY OTHER WARRANTY IMPLIED BY CUSTOM, USAGE OR COURSE OF DEALING.

Liability of MAC Medical under this warranty is limited to the repair and/or replacement of any products. MAC Medical specifically excludes and disclaims any responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative has any authority to change or enlarge the above warranty or MAC Medical liability. This document supersedes all previous warranty provisions.

To place an order, contact our customer service department at **618-476-3550** or **877-828-9975** or by email at sales@macmedical.com

MADE IN THE USA



INSPIRED
TO BE BETTER

Customer needs are our first priority.

MAC Medical, Inc.
Printed in USA
Publication No. MAN-044 Rev B
August 2021

Information regarding this product is
subject to change without prior notice.



Corporate Office
200 Admiral Weinel Blvd.
Columbia, Illinois 62236

Manufacturing Plant
820 South Mulberry Street
Millstadt, IL 62260

Phone: 618-476-3550 • Toll Free: 877-828-9975 • www.macmedical.com