




INSTRUCTION MANUAL

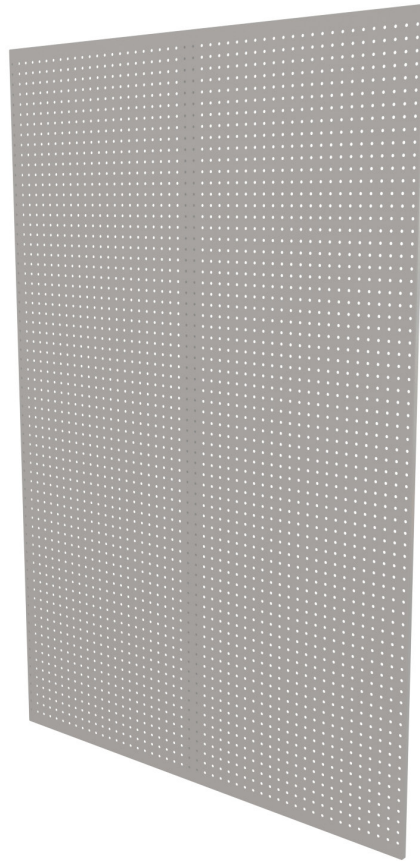

STAINLESS STEEL PEG BOARDS

USAGE AND MAINTENANCE



Read and understand all of the instructions and safety information in this manual before operating this product.

DESCRIPTION OF PRODUCT


The products covered in this manual are:
Stainless Steel Peg Boards.

PURPOSE OF THIS MANUAL

This manual covers the description, surface care and cleaning of the Stainless Steel Peg Board.

Standard Features

- Stainless steel construction (3/4" overall thickness from wall)
- 9/32" diameter holes on 1" center
- Universal key hole slots for mounting horizontally or vertically (mounting hardware not included)

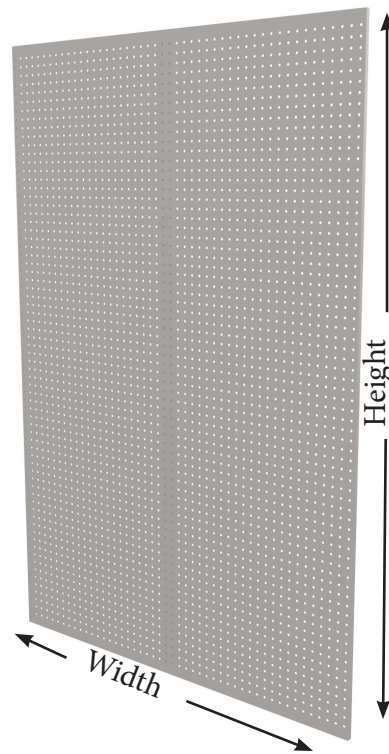

Model #	Overall Dimensions
SSPEG2436	24" W x 36" H
SSPEG3636	36" W x 36" H
SSPEG3648	36" W x 48" H
SSPEG4248	42" W x 48" H
SSPEG4848	48" W x 48" H
SSPEG4872	48" W x 72" H
SSPEG4896	48" W x 96" H

KEEP THIS MANUAL

Standard Hooks (sold separately)



Model #	Hook Size
IC0128-01	3-3/4" Stainless Steel Peg Board Hook (fits 3/16" and 1/4" diameter holes)
IC0128-02	5-3/16" Stainless Steel Peg Board Hook (fits 3/16" and 1/4" diameter holes)
IC0128-03	8-3/4" Stainless Steel Peg Board Hook (fits 3/16" and 1/4" diameter holes)
IC0901	8" Stainless Steel Peg Board Hook (fits 1/4" diameter holes)
IC0128-05	9" Stainless Steel Peg Board Hook (fits 1/4" diameter holes)

Unpacking and Inspection Instructions



CAUTION: The Stainless Steel Peg Board is heavy and it is recommended that at least two people assist with the unpacking.

- If the shipping container appears to be damaged in any way, contact the shipping company immediately. Save all damaged packaging materials.
- Carefully inspect the Peg Board while unpacking. If it appears damaged or defective in any way, call your Mac Medical representative.
- Mac Medical Stainless Steel Peg Boards are shipped fully assembled and ready for immediate use. No assembly is required.

Cleaning and Maintenance

Stainless Steel Peg Boards must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the Stainless Steel Peg Board, make sure to use the proper cleaning agents and cleaning materials.



CAUTION: DO NOT USE the following materials on the stainless steel surfaces of the Stainless Steel Peg Boards:

- Abrasive Pads
- Scrapers
- Steel Wool
- Wire Brushes



CAUTION: DO NOT USE the following agents on the stainless steel surfaces of the Stainless Steel Peg Boards:

- Hard water.
- Bleach or any products containing Chlorine
- Hydrochloric Acid
- Denatured Alcohol



NOTE: Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the Stainless Steel Peg Boards.


Use the following cleaning materials and agents:

- Soft, clean lint-free cloth
- Distilled water
- Distilled water with mild detergent
- Cleaners approved for use on stainless steel

How to Clean the Stainless Steel Peg Board

Stainless Steel:

1. Use a commercially prepared stainless steel cleaner and follow the manufacturer's directions.
2. Clean the Stainless Steel Peg Board with the grain of the Panel.

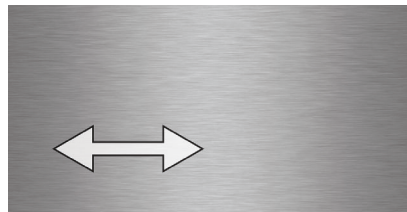

Decals or Printed Labels:

3. Use distilled water and a mild detergent applied with a lint-free cloth. **DO NOT** use any chemical cleaners.

Warranty Information



Limited Lifetime Warranty

MAC Medical warrants to the original purchaser that its Proprietary Products will be free from defects in workmanship or materials under normal use and service for the life of the product, so long as owned by the original purchaser, according to the limitations set forth below. Defective products shall be repaired or replaced at MAC Medical's option at no cost to the original purchaser provided:

1. The customer must obtain a written return authorization supplied by MAC Medical's customer service department. The product must be returned, properly packaged, with a copy of the original sales receipt and copy of the RMA authorization. Please call (877) 828-9975 or (618) 476-3550 to receive a return authorization.
2. The customer must pre-pay freight charges to and from MAC Medical and must ship merchandise properly packaged, in a way the product will not be damaged during transit to the factory. MAC Medical does not warranty any freight damage to or from the factory. It is the customer's responsibility to inspect the product for packaging damage before signing the BOL.

This warranty does not apply to products, which have been subject to abuse, misuse, accident, modification, alteration, tampering, negligence, lack of routine maintenance or misapplication; or products that have been repaired by other than MAC Medical or its authorized representatives.

This warranty does not apply to glass, fabrics, vinyl, seat coverings, cushions, padding or their stitching, gluing or installation. Component parts not manufactured by MAC Medical, such as casters, caster inserts, any components made from rubber or plastic, circuit boards, fan heaters, plumbing parts, electrical switches and other components are also excluded. For these component parts, MAC Medical will pass on the original manufacturer's warranty to MAC Medical equipment original purchasers. Please consult factory for questions regarding the warranties of these component parts. This warranty does not apply to custom fabricated products. Consult factory.

UNLESS EXPRESSLY SET FORTH IN THIS WARRANTY, THERE ARE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND / OR MERCHANTABILITY OR ANY OTHER WARRANTY IMPLIED BY CUSTOM, USAGE OR COURSE OF DEALING.

Liability of MAC Medical under this warranty is limited to the repair and / or replacement of any products. MAC Medical specifically excludes and disclaims any responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative has any authority to change or enlarge the above warranty or MAC Medical liability. This supersedes all previous warranty provisions.

To place an order, contact our customer service department at **618-476-3550** or **877-828-9975** or by email at sales@macmedical.com

MADE
IN THE **USA**

INSPIRED
TO BE BETTER

Customer needs are our first priority.

MAC Medical, Inc.
Printed in USA
Publication No. MAN-048 Rev C
May 2022

Information regarding this product is subject to change without prior notice.



Corporate Office
200 Admiral Weinel Blvd.
Columbia, Illinois 62236

Manufacturing Plant
820 South Mulberry Street
Millstadt, IL 62260

Phone: 618-476-3550 • Toll Free: 877-828-9975 • www.macmedical.com