



# INSTRUCTION MANUAL



## ONE AND TWO STEP STEP STOOLS

### Usage and Maintenance



Read and understand all of the instructions and safety information in this manual before operating this product.

## Table of Contents

Standard Features.....	3
Safety Precautions.....	4
Connecting Single Step Stools to Create a Larger Platform.....	5
Replacement Parts - General.....	6
Cleaning and Maintenance.....	7
Limited Lifetime Warranty.....	8

### DESCRIPTION OF PRODUCT

The products covered in this manual are:  
The Single Step and Two Step Stainless Steel Step Stools, and optional roll-around transport cart.

### PURPOSE OF THIS MANUAL

This manual covers the use, cleaning and maintenance of the Single Step and Two Step Step Stools.

***KEEP THIS MANUAL***



Warning! Read and understand these instructions before using the Single or Two Step Step Stools. If Step Stools are used improperly, personal injury and damage to the Step Stools could result.




Warning! If components are damaged or missing, contact your MAC Medical representative immediately. Do not use substitute parts. Use only MAC Medical replacement parts. Failure to use the correct replacement parts could lead to personal injury and damage to the Step Stools.

## Standard Features

- Heavy-duty, Stainless Steel Construction
- Tip Resistant
- Corrugated, Black Rubber Top Surface
- Rubber Floor Tips
- Single Step Stools are Stackable, and can connect together to form a large platform.

Model #	Overall Dimensions	Weight Capacity
STS121805 (Single Step)	12" D x 18" W x 6.25" H	650 lb
STS121808 (Single Step)	12" D x 18" W x 8" H	650 lb
STS121808-HD (Single Step w/ Handle)	12" D x 18" W x 8" H	650 lb
STS263015 (Two Step)	26" D x 30" W x 15" H	350 lb
STS121805-MRI (Single Step)	12" D x 18" W x 6.25" H	650 lb
STS241616-2SMRI (Two Step)	16" D x 24" W x 16" H	350 lb

Corrugated Black Rubber Surface

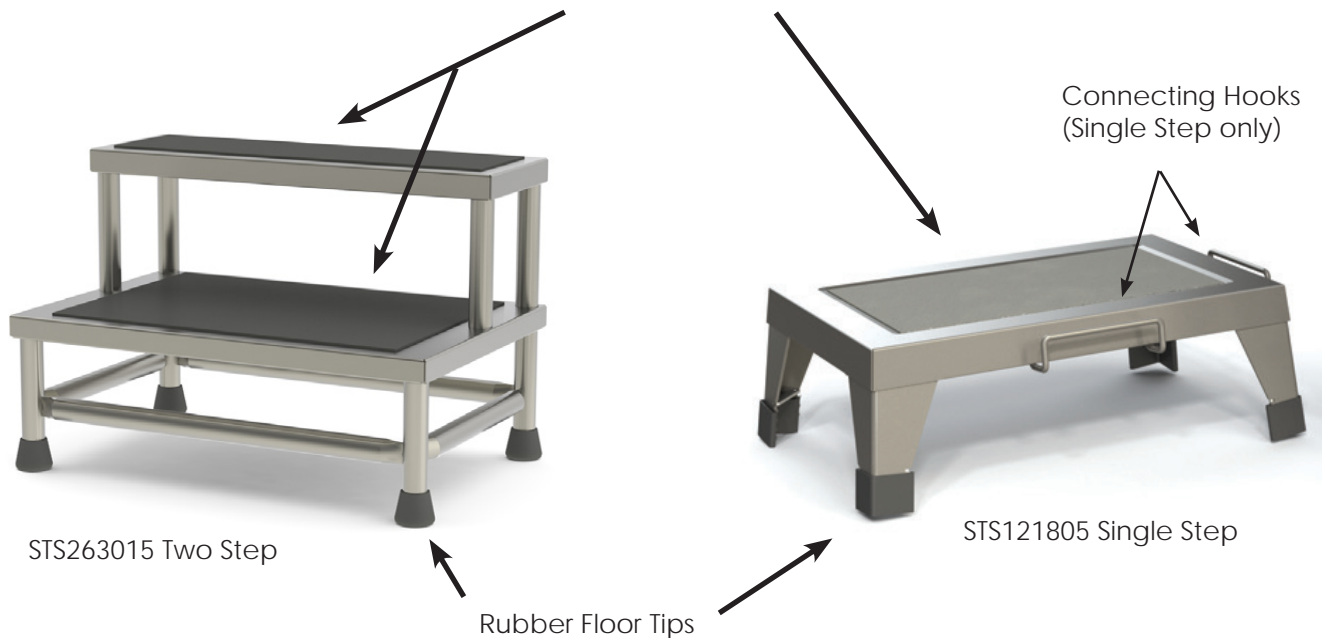

## Optional Feature

- Easy-roll transport cart with four swiveling casters (two locking), available for Single Step Stools (can hold up to five Single Step Stools).



T0800 - Optional Easy-roll Cart (for Single Step Stools Only)

## Safety Precautions



**CAUTION!** To prevent injury, please follow the precautions listed below before using either the Single or Two Step Step Stools. Practicing these safety steps can greatly increase your odds of not being injured.

- Always make sure that the surface you place the Step Stool on is level. If all four feet are not stable, the risk of a fall-related injury is greater.
- Make sure the Step Stool is in the best position to perform the work. If you must reach out or bend excessively, stop and reposition the Step Stool. Additionally, stand near the middle of the stool as much as possible.
- Do not move the Step Stool while you are on it. Always descend the stool before repositioning it.
- Never attempt to increase the height of the Step Stool by placing it on top of boxes or other equipment.
- Inspect the Step Stool for damage regularly. Cracked steps can result in the total collapse of the Step Stool. Also, ensure that all four rubber floor tips are present.
- Do not use the Step Stool on wet, icy or slippery surfaces.
- Children should not be left unsupervised around the Step Stool.

## Connecting Single Step Stools to Create a Larger Platform

- Using the connecting hooks located on one long side and one short side of the Single Step Stool, it is possible to connect a number of these stools in a variety of configurations in order to form a larger working platform.
- All necessary Safety precautions should still be observed when using the Single Step Stools in any configuration.



Fig. 1: Single Step with Hooks/Carry Handles

Single Step Stool with connecting hooks/Carry Handles (Fig. 1).



Fig. 2: Hook Steps together

Hook the lower lip of the long or short hookless side of one Single Step Stool to the desired hook on a companion Single Step Stool (Fig. 2).

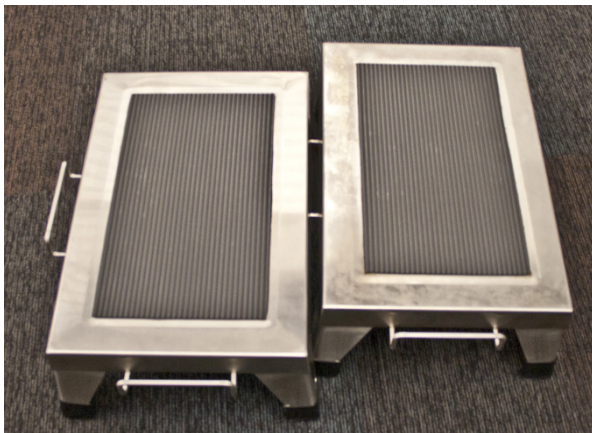


Fig. 3: Long Sides connected

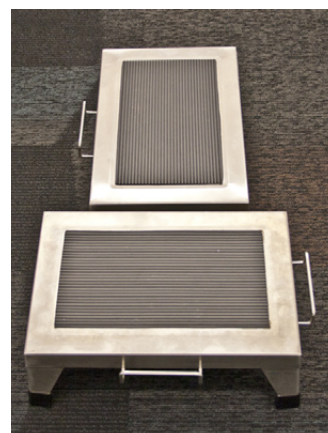


Fig. 4: Long and Short Side connected

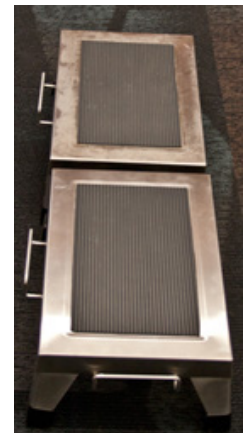


Fig. 5: Short Sides Connected

Shown are possible configurations of connected Single Step Stools (Figs. 3, 4, 5).

**WARNING:** Note that in Figure 3, when connecting along the long axis of both stools, the legs must be offset slightly. Hooking the stools together with their legs adjacent will result in a wobbly and unsafe working platform.

## Replacement Parts - General

The Step Stool replacement parts listed on this page have been identified by MAC Medical as serviceable by facility personnel and are available for purchase. To obtain MAC Medical certified parts and authorized services, contact your MAC Medical representative.



Item ID	Part Number	Description	Number	Model
1	H0090-05-04	Angle Cap - Black	4	STS121805

## Cleaning and Maintenance

Step Stools must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the Step Stools, make sure to use the proper cleaning agents and cleaning materials.



**CAUTION: DO NOT USE** the following materials on the stainless steel surfaces of the Step Stools:

Abrasive Pads  
Scrapers  
Steel Wool  
Wire Brushes



**CAUTION: DO NOT USE** the following agents on the stainless steel surfaces of Step Stools:

Hard water  
Bleach or any products containing Chlorine  
Hydrochloric Acid  
Denatured Alcohol



**NOTE:** Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the Step Stools.

Use the following cleaning materials and agents:

Soft, clean lint-free cloth  
Distilled water  
Distilled water with mild detergent  
Cleaners approved for use on stainless steel

### ***How to Clean the Step Stools***

- Using a damp, lint-free cloth and approved cleaner, wipe down all the surfaces of the Step Stool.
- Use any mild detergent to clean the corrugated rubber surfaces.
- Use a lint-free dry cloth to dry the Step Stool, or let it air dry. If air drying, ensure that the Step Stools are completely dry before storing, and store only in dry conditions.

Decals or Printed Labels:

- Use distilled water and a mild detergent applied with a lint-free cloth. **DO NOT** use any chemical cleaners.

## Limited Lifetime Warranty

MAC Medical warrants to the original purchaser that its Proprietary Products will be free from defects in workmanship or materials under normal use and service for the life of the product, so long as owned by the original purchaser, according to the limitations set forth below.

Defective products shall be repaired or replaced at MAC Medical's option at no cost to the original purchaser provided:

1. The customer must obtain a written return authorization supplied by MAC Medical's customer service department. The product must be returned, properly packaged, with a copy of the original sales receipt and copy of the RMA authorization. Please call (877) 828-9975 or (618) 476-3550 to receive a return authorization.
2. The customer must pre-pay freight charges to and from MAC Medical and must ship merchandise properly packaged, in a way the product will not be damaged during transit to the factory. MAC Medical does not warranty any freight damage to or from the factory. It is the customer's responsibility to inspect the product for packaging damage before signing the BOL.

This warranty does not apply to products, which have been subject to abuse, misuse, accident, modification, alteration, tampering, negligence, lack of routine maintenance or misapplication; or products that have been repaired by other than MAC Medical or its authorized representatives.

This warranty does not apply to glass, fabrics, vinyl, seat coverings, cushions, padding or their stitching, gluing or installation. Component parts not manufactured by MAC Medical, such as casters, caster inserts, any components made from rubber or plastic, circuit boards, fan heaters, plumbing parts, electrical switches and other components are also excluded. For these component parts, MAC Medical will pass on the original manufacturer's warranty to MAC Medical equipment original purchasers. Please consult factory for questions regarding the warranties of these component parts. This warranty does not apply to custom fabricated products. Consult factory.

UNLESS EXPRESSLY SET FORTH IN THIS WARRANTY, THERE ARE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND/OR MERCHANTABILITY OR ANY OTHER WARRANTY IMPLIED BY CUSTOM, USAGE OR COURSE OF DEALING.

Liability of MAC Medical under this warranty is limited to the repair and/or replacement of any products. MAC Medical specifically excludes and disclaims any responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative has any authority to change or enlarge the above warranty or MAC Medical liability. This document supersedes all previous warranty provisions.

To place an order, contact our customer service department at **618-476-3550** or **877-828-9975** or by email at [sales@macmedical.com](mailto:sales@macmedical.com)

**MADE IN THE USA**



INSPIRED  
TO BE BETTER

Customer needs are our first priority.

MAC Medical, Inc.  
Printed in USA  
Publication No. MAN-049 Rev C  
August 2021

Information regarding this product is subject to change without prior notice.



**Corporate Office**  
200 Admiral Weinel Blvd.  
Columbia, Illinois 62236

**Manufacturing Plant**  
820 South Mulberry Street  
Millstadt, IL 62260

Phone: 618-476-3550 • Toll Free: 877-828-9975 • [www.macmedical.com](http://www.macmedical.com)