



INSTRUCTION MANUAL



SOLUTION STANDS

Cleaning and Maintenance



Read and understand all of the instructions and safety information in this manual before operating this product.

DESCRIPTION OF PRODUCT


The products covered in this manual are:
All models of Solution Stands.

PURPOSE OF THIS MANUAL

This manual covers the description, surface care and cleaning of Solution Stands.

Standard Features

- Fully welded stainless steel construction
- Single and Double Basin designs available
- Available with Lower Shelf
- Open Frame design allows accessibility from any side
- 2" Swivel Casters

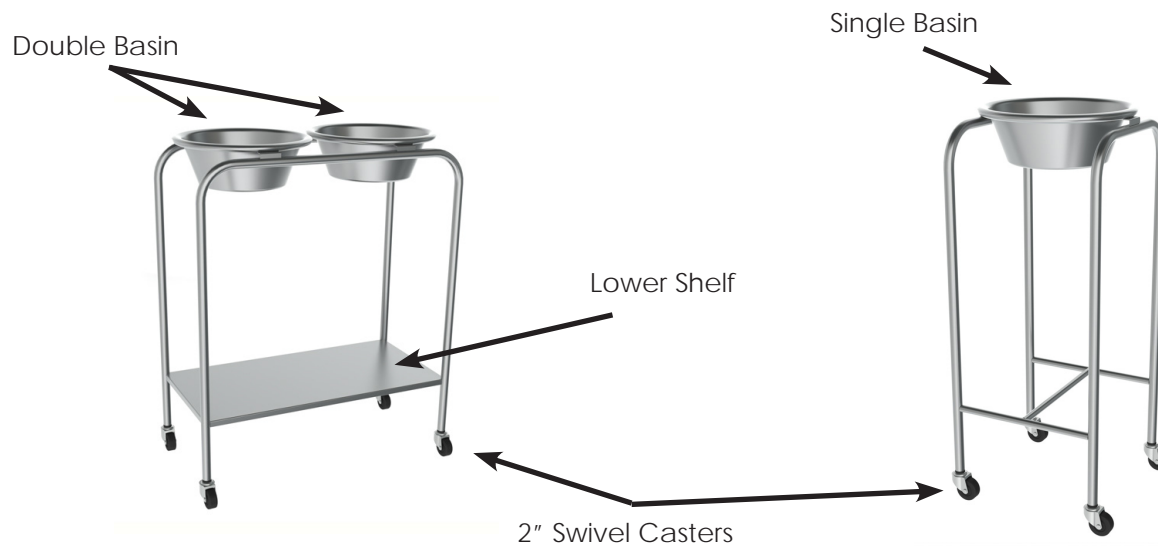

Models and Dimensions

Model	Dimensions	# of Basins	Basin Capacity	Lower Shelf
SOL-1000	14" D x 14" W x 34" H	1	7 Quart	N
SOL-1000-8.5	14" D x 14" W x 34" H	1	9 1/2 Quart	N
SOL-1001	14" D x 14" W x 34" H	1	7 Quart	Y
SOL-1001-8.5	14" D x 14" W x 34" H	1	9 1/2 Quart	Y
SOL-2000	28" D x 14" W x 34" H	2	7 Quart	N
SOL-2001-8.5	28" D x 14" W x 34" H	2	9 1/2 Quart	N
SOL-2001	28" D x 14" W x 34" H	2	7 Quart	Y
SOL-2001-8.5	28" D x 14" W x 34" H	2	9 1/2 Quart	Y

KEEP THIS MANUAL

Unpacking and Inspection Instructions



CAUTION: The Mac Medical Solution Stands are heavy, and it is recommended that two people assist in the unpacking.

- If the shipping container appears to be damaged in any way, contact the shipping company immediately. Save all damaged packaging materials.
- Carefully inspect the Solution Stand while unpacking. If any damaged or defective parts are seen, call your Mac Medical representative.
- Mac Medical Solution Stands are shipped fully assembled and ready for immediate use. No assembly is required.

Casters and Wheels - Care and Maintenance

Users of the Solution Stands are responsible for the proper care, use and maintenance of the equipment. The Solution Stand can become damaged and unsafe if used improperly.

- Never overload the Solution Stand.
- Do not drop heavy loads on the Solution Stand.
- Do not use the Solution Stand at high speeds.

Using the Solution Stand in any of the above conditions can result in failure of the Casters, Wheels or other parts of the equipment.

Periodically inspect the bottom of the Solution Stand and look for any of the following issues –

- Broken welds
- Loose nuts or bolts. These should be tightened.
- Frame Distortion. This could be caused by an overloaded Stand or by impacts with other objects while moving the Stand.

Casters:

- Make sure the Casters are in good condition. For Swivel Casters, if the caster has excessive play due to wear, it may need to be replaced. If it does not turn freely, corrosion or dirt may also force its replacement.

Wheels:

- Wheels should be checked for visible tread wear. Flat spots due to accumulation of material, such as string or thread, can cause the wheel to bind. The axle bolt nut should be removed and the foreign material removed. If the internal components are intact, the caster can be reassembled.
- Rubber tires can be badly worn and cause erratic steering, bumping, load shifting and floor damage. Replacement wheels and casters should be kept on hand to keep equipment in working order.

Cleaning and Maintenance

Stainless Solution Stands must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the Solution Stands, make sure to use the proper cleaning agents and cleaning materials.



CAUTION: DO NOT USE the following materials on the stainless steel surfaces of the Solution Stands:

Abrasive Pads
Scrapers
Steel Wool
Wire Brushes



CAUTION: DO NOT USE the following agents on the stainless steel surfaces of Solution Stands:

Hard water
Bleach or any products containing Chlorine
Hydrochloric Acid
Denatured Alcohol



NOTE: Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the Solution Stands.

Use the following cleaning materials and agents:

Soft, clean lint-free cloth
Distilled water
Distilled water with mild detergent
Cleaners approved for use on stainless steel

How to Clean a Stainless Steel Solution Stands

- Using a damp, lint-free cloth and approved cleaner, wipe down all stainless steel surfaces of the Solution Stand
- Use a lint-free dry cloth to dry the Solution Stand, or let air dry. If air drying, ensure Solution Stand is completely dry before storing. Store in a cool, dry place.

Replacement Parts - General

The Solution Stand replacement parts listed on this page have been identified by MAC Medical as serviceable by facility personnel and are available for purchase. To obtain MAC Medical certified parts and authorized services, contact your MAC Medical representative.



ITEM ID	Part Number	Description	QTY	Models
1	SOL0001	7 Qt. Basin	1 or 2	SOL-1000, SOL-1001, SOL-2000, SOL-2001
1	SOL0016	9 1/2 Qt. Basin	1 or 2	SOL-1000-8.5, SOL-1001-8.5, SOL-2000-8.5, SOL-2001-8.5
2	T0009	2" Swivel Casters	4	All

Warranty Information



Limited Lifetime Warranty

MAC Medical warrants to the original purchaser that its Proprietary Products will be free from defects in workmanship or materials under normal use and service for the life of the product, so long as owned by the original purchaser, according to the limitations set forth below. Defective products shall be repaired or replaced at MAC Medical's option at no cost to the original purchaser provided:

1. The customer must obtain a written return authorization supplied by MAC Medical's customer service department. The product must be returned, properly packaged, with a copy of the original sales receipt and copy of the RMA authorization. Please call (877) 828-9975 or (618) 476-3550 to receive a return authorization.
2. The customer must pre-pay freight charges to and from MAC Medical and must ship merchandise properly packaged, in a way the product will not be damaged during transit to the factory. MAC Medical does not warranty any freight damage to or from the factory. It is the customer's responsibility to inspect the product for packaging damage before signing the BOL.

This warranty does not apply to products, which have been subject to abuse, misuse, accident, modification, alteration, tampering, negligence, lack of routine maintenance or misapplication; or products that have been repaired by other than MAC Medical or its authorized representatives.

This warranty does not apply to glass, fabrics, vinyl, seat coverings, cushions, padding or their stitching, gluing or installation. Component parts not manufactured by MAC Medical, such as casters, caster inserts, any components made from rubber or plastic, circuit boards, fan heaters, plumbing parts, electrical switches and other components are also excluded. For these component parts, MAC Medical will pass on the original manufacturer's warranty to MAC Medical equipment original purchasers. Please consult factory for questions regarding the warranties of these component parts. This warranty does not apply to custom fabricated products. Consult factory.

UNLESS EXPRESSLY SET FORTH IN THIS WARRANTY, THERE ARE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND / OR MERCHANTABILITY OR ANY OTHER WARRANTY IMPLIED BY CUSTOM, USAGE OR COURSE OF DEALING.

Liability of MAC Medical under this warranty is limited to the repair and / or replacement of any products. MAC Medical specifically excludes and disclaims any responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative has any authority to change or enlarge the above warranty or MAC Medical liability. This supersedes all previous warranty provisions.

To place an order, contact our customer service department at **618-476-3550** or **877-828-9975** or by email at sales@macmedical.com

MADE USA
IN THE

INSPIRED
TO BE BETTER

Customer needs are our first priority.

MAC Medical, Inc.
Printed in USA
Publication No. MAN-073 Rev B
September 2021

Information regarding this product is subject to change without prior notice.



Corporate Office
200 Admiral Weinel Blvd.
Columbia, Illinois 62236

Manufacturing Plant
820 South Mulberry Street
Millstadt, IL 62260

Phone: 618-476-3550 • Toll Free: 877-828-9975 • www.macmedical.com